

NETTI BACK CUSHION

Outer and inner measurements

The use of an adjustable Velcro back has a significant effect on the inner front width. A more curved adjustable back velcro will force the side supports to move more inwards, since the back cushion are flexible.

Measurement

The green line is the measure of the outer edge to the outer edge of the cushion = dimension between the armrests.

Please note: This is different for Netti 4U and the Netti III Family as the distance between the arm support is about 30 mm wider on a Netti III.

The lower red line shows the inner/inner dimension at the top of the side support.

The upper red line shows the measurement inside/inside at the base of the side support.




Netti 4U CE COMFORT



Measurement on a SW 45 cm chair




Netti Uno Back

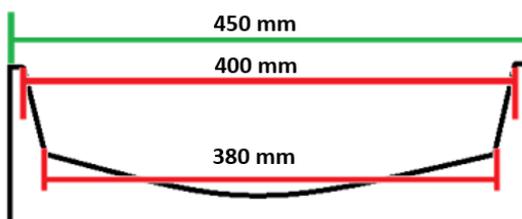

Netti Smart



Netti Stabil

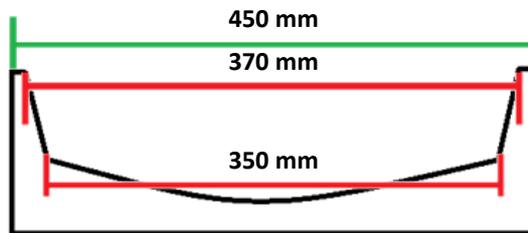

Netti Super Stabil



Netti Kyphotic

