

# MOUNTING DESCRIPTION

Replacement of  
Foot plate lock  
kit 80693

*Enable joy of life*

# NETTI WHEELCHAIRS

## REPLACEMENT OF FOOT SUPPORT FOOTPLATE LOCK

### REPLACEMENT KIT 80963




All Netti Footsupports have the same foot-plate lock

The function of the foot-plate lock:

Push the red lock forward.


It is released from the bolt underneath - sitting on the left footplate, and you can swing the right footplate up.

The left footplate is now free to be lifted too.

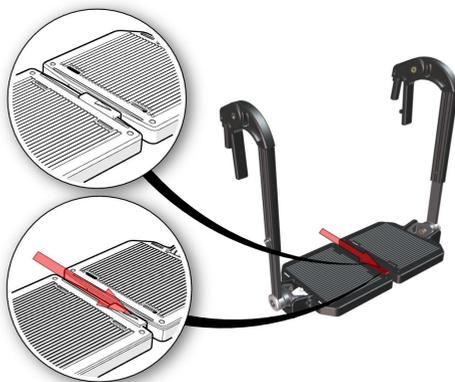


## COMPONENTS DELIVERED IN REPLACEMENT KIT 80693

1. Seegerring
2. Footplate lock-bolt
3. Plastic bolt
4. Spring
5. Lock unit

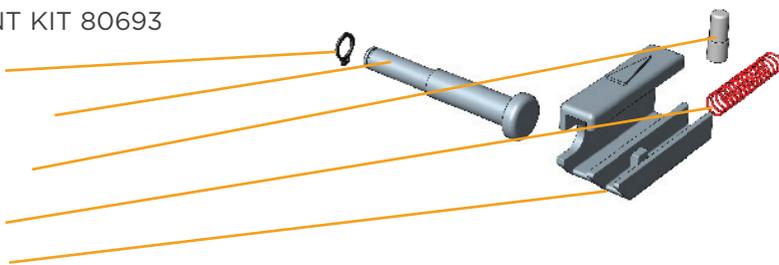


Start lock replacement with the left footplate.

Make sure all old lock parts are removed.

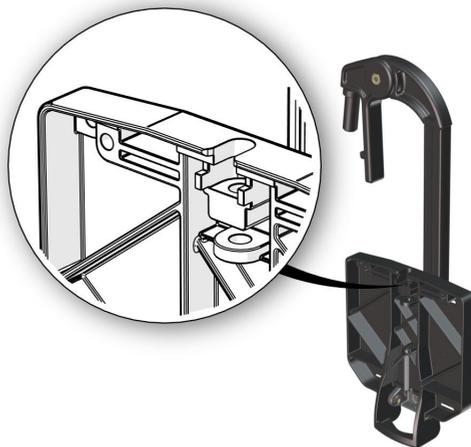

Insert the footplate lock-bolt

Fix it by clipping the seegerring to the bolt end, using a Seegerring pleyer to open the seegerring.

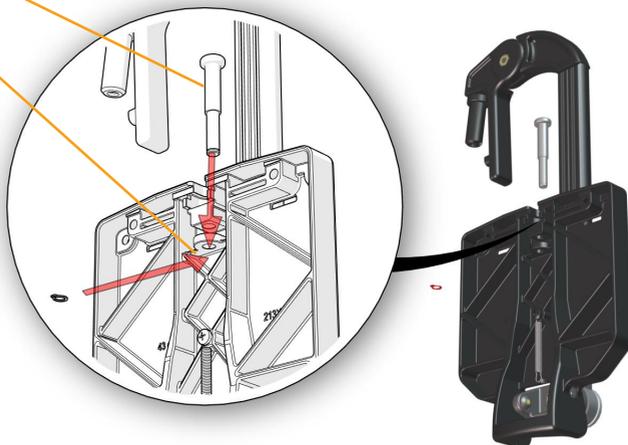



Left footplate lock-bolt fixed and ready for use.




## REPLACEMENT OF THE RIGHT PART OF THE FOOT-PLATE LOCK:

Make sure all old lock parts are removed.

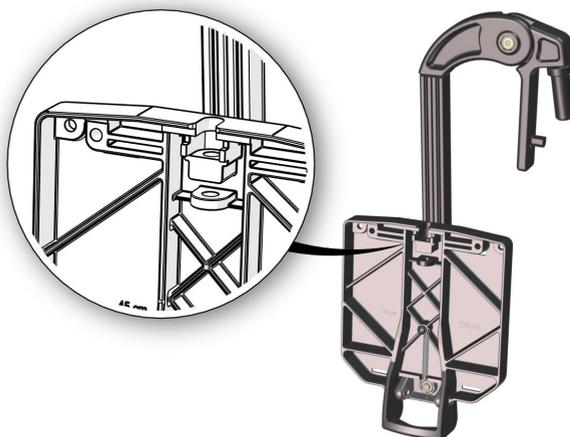

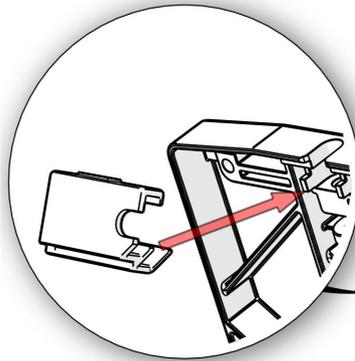

Position the red lock unit under the foot-plate to have the pointing edge hooked behind only one of the noses on the foot-plate.

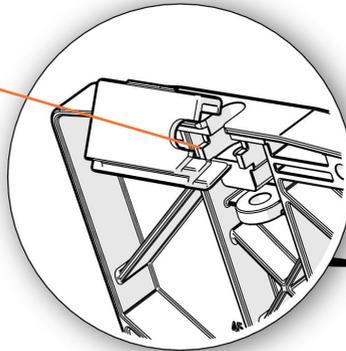


Push the red lock firm against the footplate to have it snap over the edge ( see the red arrow) and into the groove on the top of the footplate.




This may require a firm blow. The lock-unit is flexible and become more flexibel by warming up - for instance with warm water. It will require less effort to have the lock-unit snap over the edge when it is warm.




Picture shows correct position of the lock on top of the footplate.

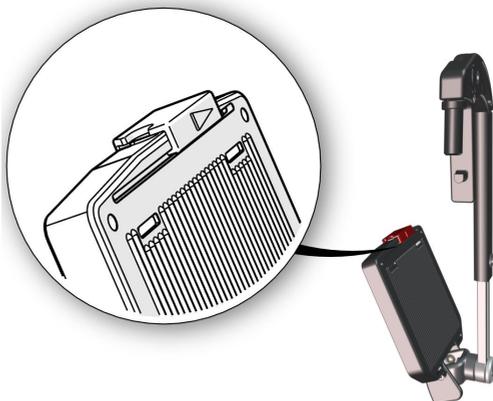

Insert the spring into the tunnel under the lockunit on the underside of the foot plate.

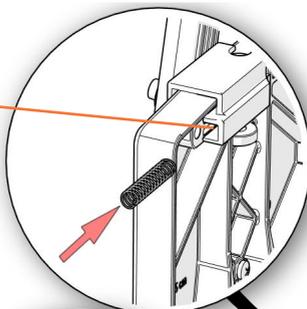

Press the plastic bolt into the hole on the footplate where the spring ends. A firm blow with a hammer fixes it permanently.

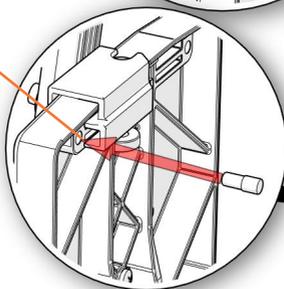

Pull the lock a couple of times to control that it snaps back into lock position.



The right side of the foot-plate lock is complete.



Mount the foot supports to the wheelchair you have and make sure the two sides of the foot-plate lock meets and locks.

The footplate lock connect the footplates and makes a stabil and firm platform for the feet of the user.





IN DIALOGUE WE CREATE  
SIMPLE SOLUTIONS AND  
ENABLE JOY OF LIFE



This product conforms to  
93/42/EEC for medical products.

2015-01 UK

FEEL FREE TO CONTACT US:



**Netti**<sup>®</sup>  
BY ALU REHAB

Alu Rehab AS  
Bedriftsvegen 23  
4353 Klepp Stasjon  
NORWAY

T: +47 51 78 62 20  
post@My-Netti.com

My-Netti.com