inspire joy of life





## MOUNTING DESCRIPTION ABDUCTION BLOCK FOR NETTI ADAPTPRO

## **Netti Abduction block**

Description	Item number	for chair models
Abduction block Small Abduction block Medium Abduction block Large	92495 92496 92497	Netti AdaptPro Netti AdaptPro Netti AdaptPro
Attachment plate for all:		93866


The AdaptPro seat plate is prepared for fixing Abduction block attachment plate direct to the plate with 4 pc M5 x 10 hex socket screws.

Depending on the seat width, the bracket will fit between the velcro bands for fixing the seat cushion or it will cover part of the velcro.

Please use the 3M velcro in the abduction block kit to ad velcro on top of the bracket - if the bracket is covering some of the velcro.

See the red ellipses.

This will make sure you have the original size velcro to hold the cushion.

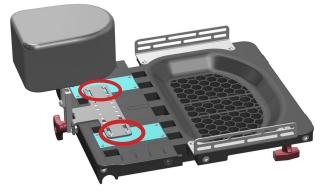

Seat width 400 mm



Seat width 350 mm

NORWAY: Alu Rehab AS | Bedriftsvegen 23 N-4353 Klepp Stasjon post@My-Netti.com Phone +47 51 78 62 20 DENMARK: Alu Rehab ApS | Kløftehøj 8 | DK-8680 Ry | T: +45 87 88 73 00 | F: +45 87 88 73 19 info@My-Netti.com |